

Performance evaluation of a rapid point-of-care microfluidic immunofluorescence assay for the detection of SARS-CoV-2 antibodies

The LumiraDx SARS-CoV-2 Antibody (Ab) Test is a point-of-care (POC) test that uses a rapid, high sensitivity microfluidic immunofluorescence assay to qualitatively detect total antibodies to severe acute respiratory syndrome coronavirus 2 (SARS-CoV-2). The test was evaluated for clinical agreement using samples with real-time polymerase chain reaction (RT-PCR)-confirmed SARS-CoV-2 infection status across POC settings.

Methods

Plasma samples from participants (≥ 18 years) were collected from clinical studies and commercial suppliers for positive and negative agreement analysis. Matrix equivalency was determined using donor whole blood, plasma and serum samples spiked with immunoglobulin (Ig) M and IgG. Cross-reactivity was analysed using samples with known related and high-prevalence pathogens. Fingertick application with capillary blood (applied directly or using transfer tube) from participants (≥ 18 years) was analysed following RT-PCR testing. Application was performed by untrained users and ease of use of the LumiraDx SARS-CoV-2 Ab Test was investigated.

Results

An overall positive agreement of 97.2% (95% confidence interval (CI): 90.4, 99.2%; $n=72$) was determined in samples obtained ≤ 118 days following RT-PCR testing, and 100% ($n=59$) in a subset obtained ≥ 7 days following RT-PCR testing. Samples from participants without a history of SARS-CoV-2

infection (endemic symptomatic, asymptomatic and non-endemic participants) showed an overall negative agreement of 100% (95% CI: 98.7, 100%; $n=290$) with RT-PCR testing. Matrix equivalency demonstrated 100% agreement and no cross-reactivity with any of the tested pathogens; 100% agreement was observed between capillary fingertick application and RT-PCR results, as well as between sample application methods. Test operators indicated that the LumiraDx SARS-CoV-2 Ab Test was easy to use.

Conclusion

The LumiraDx SARS-CoV-2 Ab Test demonstrated excellent agreement when compared with RT-PCR test results, using capillary and venous whole blood, plasma and serum. This study reported 97.2% positive and 100% negative agreement overall in samples collected after RT-PCR testing, and 100% positive agreement in samples obtained at least 7 days after RT-PCR confirmation when using the LumiraDx SARS-CoV-2 Ab Test.

The LumiraDx SARS-CoV-2 Ab Test has achieved CE mark. The data represented in this document is only applicable to the CE marked product.

LumiraDx UK Ltd: Unit 1, Block 5, Dumyat Business Park, Bond Street, Alloa, FK10 2PB, UK

Copyright © 2021 LumiraDx UK LTD. All rights reserved worldwide.

Content should be used for the use of the LumiraDx products only and in line with instructions provided. You may not, except with our express written permission, distribute or commercially exploit the content. Nor may you transmit it or store it in any other form of electronic retrieval system other than for the purpose of use of the LumiraDx Instrument or LumiraDx Test Strips. Information provided is subject to change without notice.

LumiraDx and Flame logo are trademarks of LumiraDx International LTD. Full details of these and other registrations of LumiraDx can be found at lumiradx.com/IP. All other trademarks are the property of their respective owners.

The LumiraDx SARS-CoV-2 Antibody Test has achieved CE Mark. Not all products are available in all countries and regions. Please check with your local LumiraDx sales representative or distributor for availability in specific markets.

lumiradx.com